Section 1 Section 1

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exenatide injection
250 µg/mL, 2.4 mL
exénatide injectable
250 µg/mL, 2.4 mL

AstraZeneca

Read these instructions carefully BEFORE using your BYETTA® pen. Also read the BYETTA Consumer Information sheet that comes with the BYETTA pen carton.

You need to use the pen correctly in order to get the most benefit from BYETTA. Failure to follow these instructions completely may result in a wrong dose, a broken pen or an infection.

These instructions do not take the place of talking with your healthcare professional about your medical condition or your treatment. If you are having problems using your BYETTA pen, contact your healthcare professional or AstraZeneca at 1-800-668-6000.

IMPORTANT INFORMATION ABOUT YOUR BYETTA PEN

- BYETTA is injected twice a day, the pen contains enough medicine for 30 days. You do not have to measure any doses, the pen measures each dose for you.
- **DO NOT TRANSFER THE MEDICINE IN THE BYETTA PEN TO A SYRINGE.**
- If any part of your pen appears broken or damaged, do not use the pen.
- **Do not share your pen or needles as this may risk transmission of infectious agents.**
- This pen is not recommended for use by people who are blind or who cannot see well enough to see the arrows in the dose window. Help will be needed by a person trained to use the pen.
- Healthcare professionals or other caregivers should follow local or institutional policies regarding needle handling.
- **Follow the instructions for hygienic injection technique recommended by your healthcare professional.**
- Follow Section 2 only to set up a new pen before first use.
- Section 3 of this manual should be used for every injection.

ABOUT PEN NEEDLES

What kinds of needles can be used with my BYETTA pen?

- Recommended for use with BD Pen Needles.

Do I use a new needle for each injection?

- Yes. Do not reuse needles.
- Remove the needle immediately after each injection. This will help prevent leakage of BYETTA, keep out air bubbles, reduce needle clogs, and decrease the risk of infection.
- Never push the injection button on the pen unless a needle is attached.

How do I throw away my needles?

- Throw away used needles in a puncture-resistant container or as recommended by your healthcare professional.
- Do not throw away the pen with a needle attached.

STORING YOUR BYETTA PEN

How do I store my BYETTA pen?

- Store your unused BYETTA pen in the original carton in a refrigerator at 2°C to 8°C protected from light. Do not freeze. Throw away any BYETTA pen that has been frozen.
- Once in use, your BYETTA pen should be kept at 2°C to 25°C.
- Do not store the pen with the needle attached. If the needle is left on, BYETTA may leak from the pen and air bubbles may form in the cartridge.

Keep your pen and needles out of the reach of children.

How long can I use a BYETTA pen?

- You can use your BYETTA pen for up to 30 days after setting up a new pen for first use. **After 30 days, throw away the BYETTA pen, even if it is not completely empty.**
- Mark the date when you first used your pen and the date 30 days later in the spaces below:

Date of First Use _____

Date to Throw Away Pen _____

- BYETTA should not be used after the expiration date printed on the pen label.

How do I clean my BYETTA pen?

- If needed, wipe the outside of the pen with a clean, damp cloth.
- White particles may appear on the outside tip of the cartridge during normal use. You may remove them with an alcohol wipe or alcohol swab.

Please see the accompanying complete BYETTA Consumer Information. For additional information, contact your healthcare professional or AstraZeneca at 1-800-668-6000 or visit www.astrazeneca.ca.

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Section 2 Section 2

GETTING STARTED POUR COMMENCER

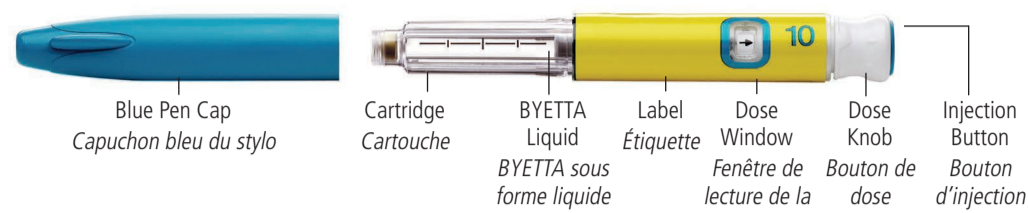
Read and follow the directions in this section only after you've read Section 1 – What You Need To Know About Your BYETTA Pen.

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
Set up your new pen just before you use it for the first time. Follow the New Pen Setup only once. For routine use, do not repeat the New Pen Setup. If you do, you will run out of BYETTA before 30 days of use.

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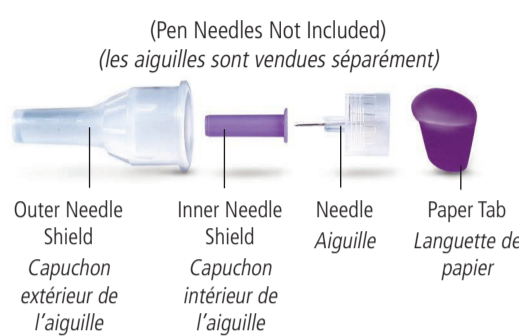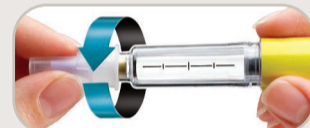
Note: A small air bubble in the cartridge is normal.

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Note: If you cannot turn the dose knob clockwise to the 10, see **Commonly Asked Questions**, number 7, in Section 4 of this user manual.

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- **PUSH & HOLD**
- Use thumb to firmly push the injection button in until it stops, then continue holding the injection button in while **slowly counting to 5**.
- If you do not see a stream or several drops come from the needle tip, repeat Steps C & D.

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Note: If you do not see liquid after 4 times, see **Commonly Asked Questions**, number 3, in Section 4 of this user manual.

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Note: If you cannot turn the dose knob, see **Commonly Asked Questions**, number 7, Section 4 of this user manual.

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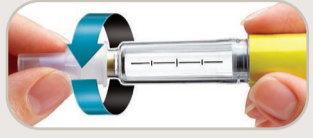
Note: A small air bubble will not harm you or affect your dose.

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Note: If you cannot turn the dose knob clockwise to the **10**, see **Commonly Asked Questions**, number 7, in Section 4 of this user manual.

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Note: If you see several drops of BYETTA leaking from the needle after the injection, the injection button was not pushed in all the way. See **Commonly Asked Questions**, number 4, in Section 4 of this user manual.

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STEP 5 ÉTAPE 5

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- Turn dose knob clockwise until it stops and the **10** is in the dose window.

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Note: If you cannot turn the dose knob, or if your pen leaks, your full dose has not been delivered. See **Commonly Asked Questions**, numbers 4 and 7, in Section 4 of this user manual.

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Section 4 Section 4

COMMONLY ASKED QUESTIONS QUESTIONS COURAMMENT POSÉES

- Do I need to do the New Pen Setup before every dose?**
 - No. The New Pen Setup is done only **once**, just before each new pen is used for the first time.
 - The purpose of the setup is to make sure that your BYETTA pen is ready to use for the next 30 days.
 - If you repeat the New Pen Setup before each routine dose, you will not have enough BYETTA for 30 days. The small amount of BYETTA used in the New Pen Setup will not affect the 30-day supply of BYETTA.

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- Why are there air bubbles in the cartridge?**
 - A small air bubble is normal. It will not harm you or affect your dose.
 - If the pen is stored with a needle attached, air bubbles may form in the cartridge. **Do not** store the pen with the needle attached.

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- What should I do if BYETTA does not come out of the needle tip after four tries during New Pen Setup?**
 - Remove the needle by carefully putting the outer needle shield back over the needle. Unscrew and dispose of properly.
 - Attach a new needle and repeat **New Pen Setup, Steps B – E**, in Section 2 of this user manual. Once you see several drops or a stream of liquid coming out of the tip of the needle, the setup is complete.

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- Why do I see BYETTA leaking from my needle after I have finished my injection?**
 - It is normal for a single drop to remain on the tip of your needle after your injection is complete. If you see more than one drop:
 - You may not have received your full dose. **Do not** inject another dose. **Consult with your healthcare professional about how to handle a partial dose.**
 - To prevent this, for your next dose, **firmly push and hold** the injection button in and **slowly count to 5** (see **Section 3, Step 4: Inject the Dose**).

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- How can I tell when the injection is complete?**
 - The injection is complete when:
 - You have firmly pushed the injection button in all the way **until it stops** and
 - You have slowly counted to 5** while you are still holding the injection button in and the needle is still in your skin and
 - The **10** is in the center of the dose window.

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- Where should I inject BYETTA?**
 - BYETTA should be injected into your abdomen, thigh, or upper arm using the injection technique recommended by your healthcare professional.

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- What if I cannot pull, turn, or push the dose knob?**
 - Check the symbol in the dose window. Follow the steps next to the matching symbol.

- If **10** is in the dose window:**
 - Pull the dose knob out until **10** appears.

- If **10** is in the dose window and the dose knob will not turn:**
 - The cartridge in your BYETTA pen may not have enough liquid to deliver a full dose. A small amount of BYETTA will always remain in the cartridge. If the cartridge contains a small amount or looks empty, obtain a new BYETTA pen.

- If **10** and part of the **10** are in the dose window and the dose knob cannot be pushed in:**
 - The dose knob was not turned all the way. Continue turning the dose knob clockwise until **10** is in the center of the dose window.

- If part of **10** and part of **10** are in the dose window and the dose knob cannot be pushed in:**
 - The needle may be clogged, bent, or incorrectly attached.
 - Attach a new needle. Make sure needle is on straight and screwed on all the way.
 - Firmly push the injection button in all the way. BYETTA should come from needle tip.

- If **10** is in the dose window and the dose knob will not turn:**
 - The injection button was not pushed in all the way and a complete dose was not delivered. **Consult with your healthcare professional about how to handle a partial dose.**
 - Follow these steps to reset your pen for your next injection:
 - Firmly push the injection button in all the way **until it stops**. Keep holding the injection button in and **slowly count to 5**. Then turn the dose knob clockwise until **10** appears in the dose window.
 - If you cannot turn the dose knob, the needle may be clogged. Replace the needle and repeat the step above.
 - For your next dose, be sure to **firmly push and hold** the injection button in and **slowly count to 5** before removing needle from skin.

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© AstraZeneca 2014
BYETTA® is a registered trademark of Amylin Pharmaceuticals LLC and the AstraZeneca logo is a registered trademark of AstraZeneca AB, used under license by AstraZeneca Canada Inc.